Prior to use, carefully read the instructions for use.

EN

EPITEX

FINISHING AND POLISHING STRIPS

For use only by a dental professional in the recommended indications.

INTENDED USE

For contouring, finishing and polishing of interproximal composite, compomer and glass ionomer restorative surfaces.

	Width	Length	Thickness	Grit	Use
Coarse (Blue)	5 mm	10 m	50 µm	#280	Contouring
Medium (Green)	5 mm	10 m	50 µm	#500	Finishing
Fine (Grey)	5 mm	10 m	50 µm	#800	Polishing
Extra Fine (Pink)	5 mm	10 m	50 µm	#1200	Polishing
Matrix strip (Transparent)	8 mm	10 m	50 µm	-	Contouring prior to light cure

- Remove excess material with a suitable bur.
- Choose a suitable grit.
- Insert the strip between the interproximal contact point. Firmly grab both ends of the strip and draw back and forth.
- Discard used strips.

CAUTION

- EPITEX has sharp edges. Wear protective gloves and eyewear when handling the strips. Take care not to injure the patient's soft tissues.
- For infection control reasons, strips are for single use only. Dispose of all waste according to local regulations.

PACKAGES

- Starter Kit
 - 1 reel (10 m) of each grit and Matrix Strip, 1 piece of dispenser stand
- Refill pack available in all grits and Matrix strip
 - 1 reel (10 m)

STORAGE

Store away from high temperature or humidity.

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